nosiboo^{*}go[®]
portable nasal aspirator

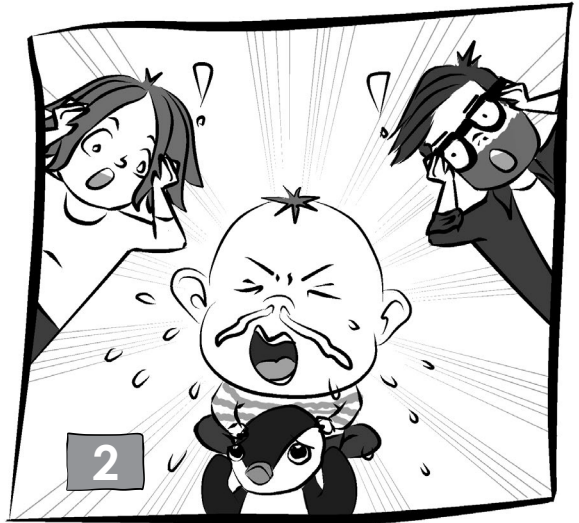
2020.09.01_Nosiboo_Go_QG_Digit_1.1

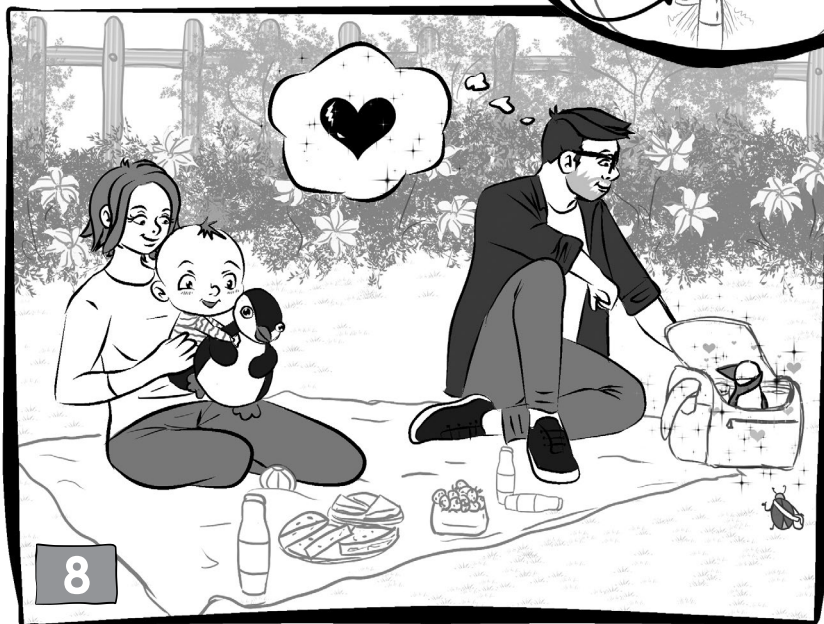
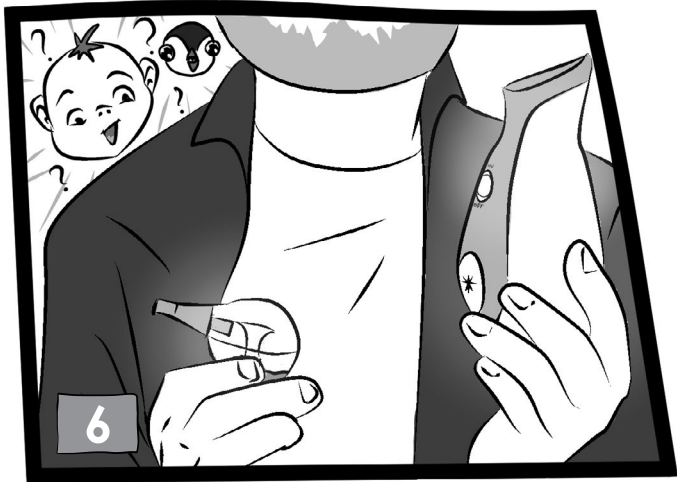
nosiboo^{*}go[®]
portable nasal aspirator

Before using the Nosiboo Go, carefully read this guide and the **User Manual** as well! Please keep this guide for future reference.


Pingo & Nosiboo

IN THE BOX

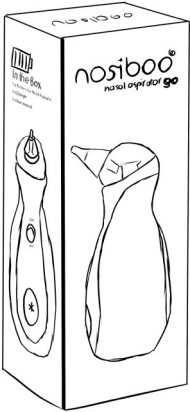Quick start guide

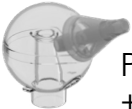
Safety instructions

Nosiboo Go


Charger

Penguin head
+Nose tip

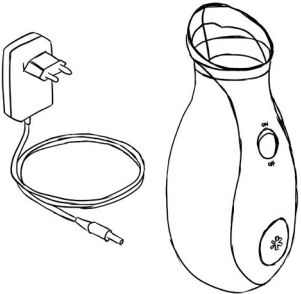Air mesh

Brush

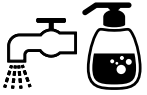
Sack

Contact the distributor if any part is missing!

USING THE NOSIBOO GO



The device should be used for removing the nasal mucus from children's nose. It should not be used for any other purposes.


First clean* and dry the parts of the Penguin head.

*Check the "CLEANING" section below for more details about the proper cleaning method.

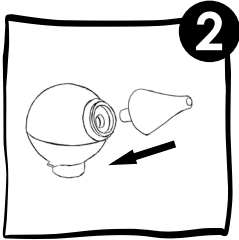
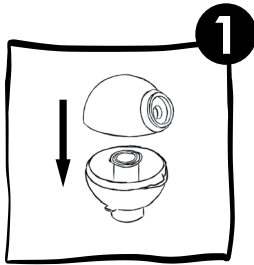
Assemble the Penguin head.

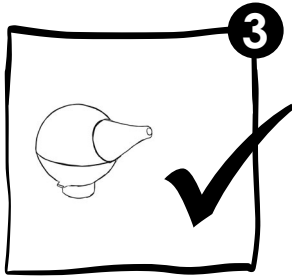Insert the nose tip into the Penguin head.

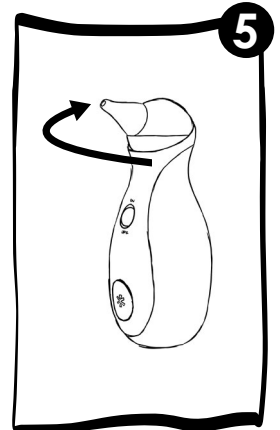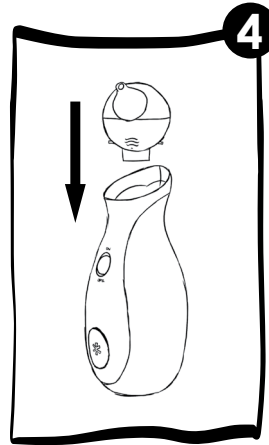
Make sure the air mesh is in its place, the Penguin head is assembled properly and installed on the device in the correct way.

The device is ready to use.

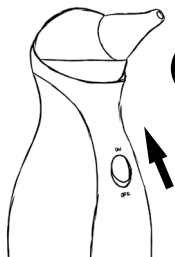INSTRUCTIONS FOR USE & SETTINGS

1 Getting started

Simply turn it on.

The effective and strong (max.) suction power has been proven safe for nasal aspiration by ENT doctors.

2 Hold your child in a stable position.

- * Aspirate the nose of infants after laying them into a stable position and hold their head during the process.
- * You may also sit them in an infant bouncer or infant seat.
- * If the child is already able to sit, sit them in your lap. In this case, do not lay the child on their back during the aspiration.

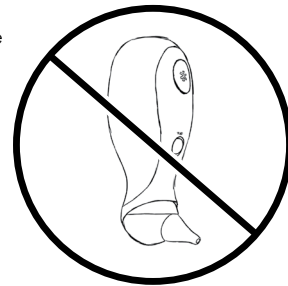
3 Insert the clean, dry nose tip into the child's nostril.

- * Move the nose tip around gently to make sure that you get all areas of the nostril.


If you experience that mucus or liquid has entered the device then stop using the Nosiboo Go, turn it off and contact the Nosiboo customer service team right away.

- * Don't turn the device upside down meanwhile use.

4

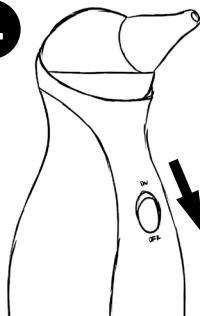
Switch off the device after use.

- * We highly recommend to remove the Penguin head and clean it right away.

5



Charging

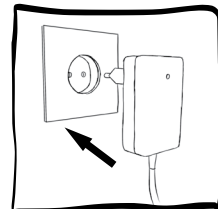- * After use, or if the device's battery just ran low, wait a few minutes until the device cools down and then start charging it.
- * Green LED - Fully charged
- * Red LED - Still charging
- * While being charged, the device cannot be used.


CLEANING

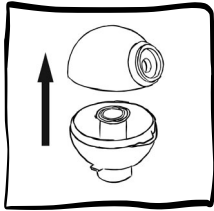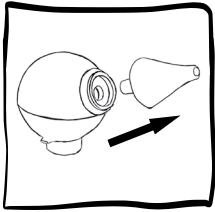Detach the Penguin head in the following order:

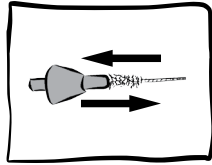* Wash the parts with liquid soap and lukewarm water.

Use the brush to clean the nose tip.

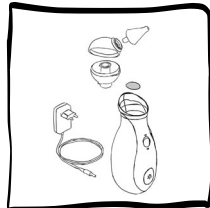
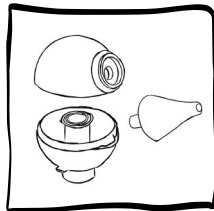
* You can soak the parts in a disinfection liquid, just make sure the liquid is safe to use for baby equipment.


* Let the parts dry completely before the next use.

* It is not allowed to use boiling water, microwave oven, vaporizer or baby bottle sterilizer for cleaning the spare parts because they might deform by heat!

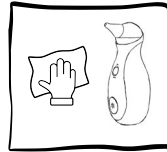- * The air mesh is there to prevent solid objects and dust from entering the housing.
- * The air mesh is not there to absorb any mucus or liquid.
- * If the air mesh is dirty or clogged, you can replace it.
- * It can function well for a very long time next to the proper usage of the device.

- * Clean the housing with a dry, soft cloth.
- * Do not use any liquids, detergents or chemicals.
- * Keep the device in a place where it is protected from dust, direct sunlight and heat.

HINTS & TIPS

- * The Penguin head must be assembled and disassembled in the right order.
- * Do not cover the vents on the back of the device while in use.
- * Dry, very thick mucus can be dissolved by a mild saline solution. After using it, wait a few minutes, wipe excess discharge with a tissue, then reuse the nasal aspirator as directed.
- * Do not use more than 5 ml of saline solution at once.

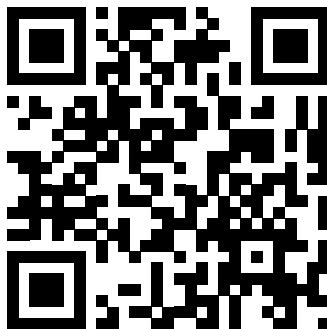
- * The device should only be used by adults.
- * Do not allow children to play with the device, it is NOT A TOY.

User manual

AFTER-SALE SERVICE

For more information about usage, cleaning, and for troubleshooting, see the User Manual.

customers in EU: nosiboo.eu
customerservice@nosiboo.com

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