
nosiboo^{*}pro[®]

electric nasal aspirator

reddot award 2014
winner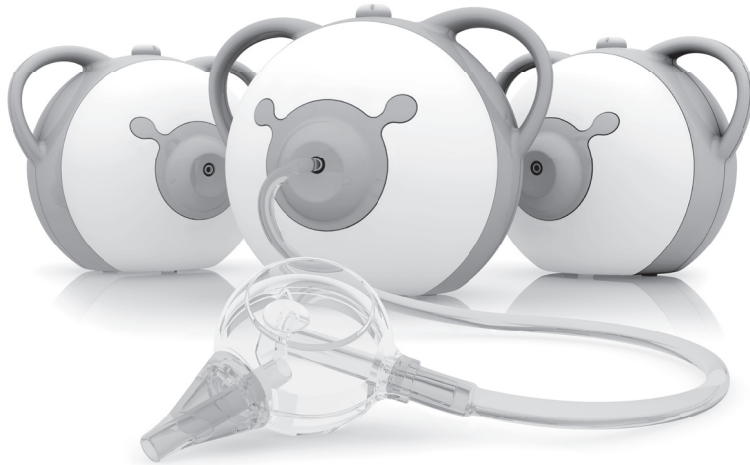

Safety instructions

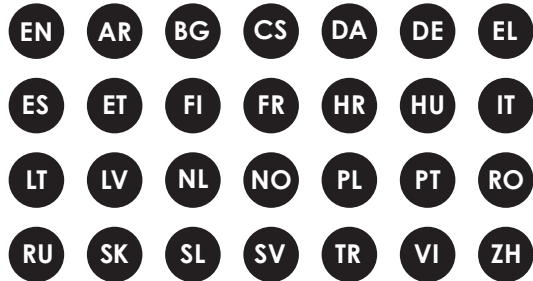
Table of contents

EN Safety instructions	FR Consignes de sécurité	RO Instrucțiuni de siguranță
AR تعليمات السلامة	HR Sigurnosne upute	RU Правила техники безопасности
BG Инструкции за безопасност	HU Biztonsági utasítások	SK Bezpečnostné pokyny
CS Bezpečnostní pokyny	IT Istruzioni di sicurezza	SL Varnostna navodila
DA Sikkerhedsanvisninger	LT Saugumo nurodymai	SV Säkerhetsinstruktioner
DE Sicherheitshinweise	LV Norādījumi par drošību	TR Güvenlik Talimatları
EL Οδηγίες ασφαλείας	NL Veiligheidsinstructies	VI Chì dãn an toàn
ES Instrucciones de seguridad	NO Sikkerhetsinstruksjoner	ZH 安全指示
ET Ohutusjuhised	PL Instrukcje dotyczące bezpieczeństwa	
FI Turvallisuusohjeet	PT Instruções de segurança	

*[®]nosiboo[®]pro

electric nasal aspirator

EN

Safety instructions

Dear Customer,

thank you for purchasing our Nosiboo Pro Electric Nasal Aspirator. Please, follow the instructions provided for your device. To obtain the very best performance for your nasal aspirator, please read the **Quick Guide** too. For more information we recommend reading the **User Manual**:

Attract Kff. is an ISO 13485 certified medical device manufacturer. This product is an MDD Legacy Device.

Actor EUDAMED No.: HU-MF-000004252

Please keep these documents for future reference.

Please read all instructions carefully before using the device.

The warranty will be only validated if the device was used according to its intended purpose and does not bear signs of damage.

Nosiboo Pro is a hygienic medical device. You may only exercise your right of withdrawal if the device is undamaged and the protective foil on the device and the nose tip is intact. For hygienic reasons, we are unable to refund the product if the protective foil has been opened.

Safety instructions

Type BF applied parts
(increased protection against electric shock)

Double insulation

Class IIa medical device

Name and address of the manufacturer

Read the Safety instructions

Read the online User manual

For home use only

GTIN – Global Trade Item Number (UDI-DI)

Serial number (Unique device identifier)

Item number (type number)

Lot number (production identifier)

Keep dry

Protection against solid particles of diameter greater than 12.5 mm; protected from condensation.

Hazardous waste

General use

Use only as directed. Misuse can be dangerous.

The device should only be used by adults.

Do not use the nasal aspirator for nasal washing!

In case of unexpected failure or malfunction, immediately switch off the device.

Do not allow children to play with the device, it is NOT A TOY. (E.g., the power cable can wind easily around a child's neck.)

Do not leave the device unattended near children.

Do not use the device in the bathroom or in any other humid or wet environment.

Do not handle the power cable and the metal parts of the device with wet hands. Do not cover the vents on the back of the device while in use.

The device is intended to be used only for removing nasal mucus from a child's nose. It shall not be used for any other purposes.

Keep the device and its accessories out of reach of children.

Make sure that all parts are properly attached before using the device.

Do not use the device without the air mesh.

Make sure that the housing, the connecting tube, and the Colibri head are not damaged or cracked.

Use the device only with the Colibri head supplied by the manufacturer.

The part of the Colibri head attached to the connecting tube shall point downwards while in use.

Do not turn the Colibri head upside down meanwhile in use.

Cleaning, maintenance and storage

Keep the device away from heat (e.g. furnace, heater, or convector) because it can cause damage to the insulation.

Do not tie knots on the power cable or the connecting tube.

To avoid damage, do not place heavy items on the cable.

A damaged power cable may cause electric shock.

Do not use the power cable and the device if they are damaged or cracked. Do not remove the mounting screws from the housing. Do not reconstruct the device for other purposes. It may be life-threatening.

When you disassemble the Colibri head for cleaning, do not leave the parts within reach of children. Children may chew or swallow the small parts.

It is not allowed to use a dishwasher, boiling water, microwave oven, vaporizer, or baby bottle sterilizer for cleaning the spare parts, because they might deform by heat!

Do not wash the housing or the power cable with water or any other liquid. Do not handle the device with wet hands. This may cause electric shock or malfunction.

Always unplug the power cable from the outlet after using the device.

The device should be stored in a cool, dry place. Avoid direct sunlight or heat.

Power source

Use the device only with the power cable supplied by the manufacturer. Failure to do so may result in fire hazard.

Be sure the plug of the power cable is compatible with your power outlet!

The device should be connected to or disconnected from the power network only if the control button is in OFF position (the device is switched off).

Specifications

Max. motor performance: 500 W
Adjustable motor performance: 200-390 W
Performance: 370 VA
Differential pressure: 0.04-0.08 bar
Max. air flow: 30-42 l/min
Power supply: 230 V, 50 Hz
IP class: IP21
Protection class: II
Type of applied parts: BF
Applied parts: the complete housing
Operation type: no continuous operation
Operation length: on 15 min/appr. off 50 min
Duration of continuous use: max 15 min.
Safety switch release time: after 15 minutes
Tripping temperature of the thermal switch: 95 °C
Average housing temperature: 30-35 °C after 1 min continuous use
Noise level: 60-70 dB
Operating temperature range: -10 °C... +40 °C
Operating relative humidity: max. 75%
Storage and transportation temperature: -20 °C...+65 °C
Storage and transportation relative humidity: 10...95%
Expected life of the housing (electrical parts): 3 years
Expected life of the Colibri head: 3 months
Diameter of the nasal insertion end of the standard nose tip:
max. 7.0 mm
Length of the nasal insertion end of the standard nose tip: max.
8.0 mm
Diameter of the nasal insertion end of the newborn's nose tip:
max. 4.9 mm
Length of the nasal insertion end of the newborn's nose tip:
max. 12.6 mm
Weight: 1090 g
Dimensions (housing, W x H x D): 198x162x172 mm
Dimensions (complete package): 180 x 180 x 225 mm
MD Category: II.a
NBOG: MD1102
MDD/MDR: MDD Legacy Device
Actor EUDAMED No.: HU-MF-000004252

We reserve the right to make technical changes.

The manufacturer shall not be held responsible for the failure of products from unauthorised import sources!

Contact

Manufacturer contact details.
Attract Kft.
H-7622 Pécs, Siklósi út 1/1., Hungary
Tel.: +36 72 551 642 (Mo–Fr: 8 am – 4 pm)
E-mail: customerservice@nosiboo.com

PRRC (Person Responsible for Regulatory Compliance):
Attract Kft. – Quality Manager
H-7622 Pécs, Siklósi út 1/1., Hungary
Tel.: +36 72 551 642 (Mo–Fr: 8 am – 4 pm)
E-mail: quality@nosiboo.com

For more information we suggest reading the user manual and our quick guide as well.

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