# nosiboo<sup>\*</sup>pro<sup>®</sup>

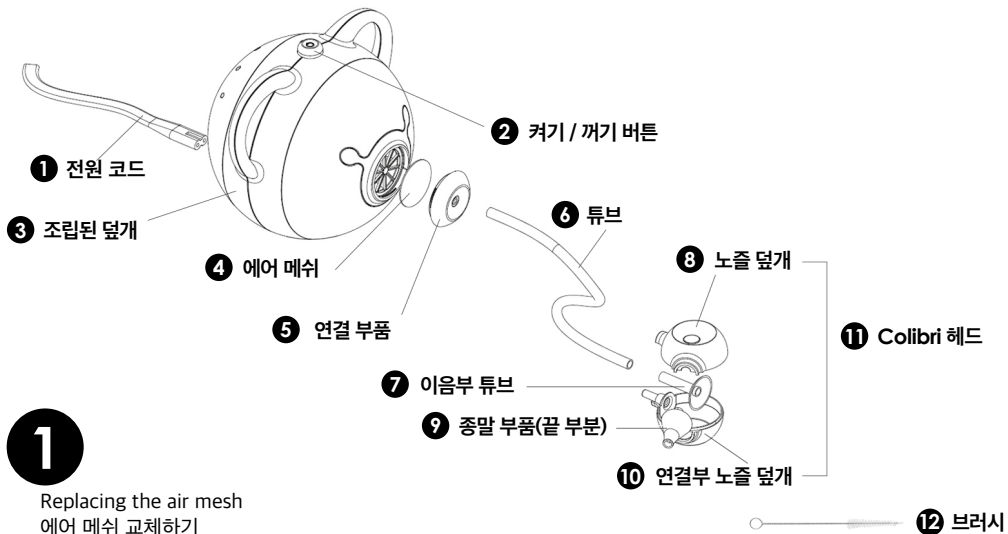
electric nasal aspirator

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
**EN** USER MANUAL

reddot award 2014  
winner

**EN** 1. Power cord 2. On/Off control button 3. Assembled housing 4. Air mesh 5. Connecting unit 6. Tube 7. Collared tube 8. Nozzle housing 9. Nose tip 10. Nozzle housing with connection 11. Colibri head 12. Cleaning brush

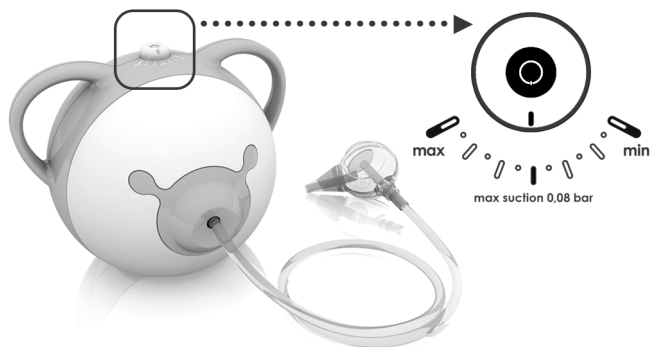**2**  
Adjusting the suction power  
흡인력 조절하기

For children under 6 months of age, you can use the nose tip designed for newborns. Above 6 months of age, it is recommended to use the standard nose tip.

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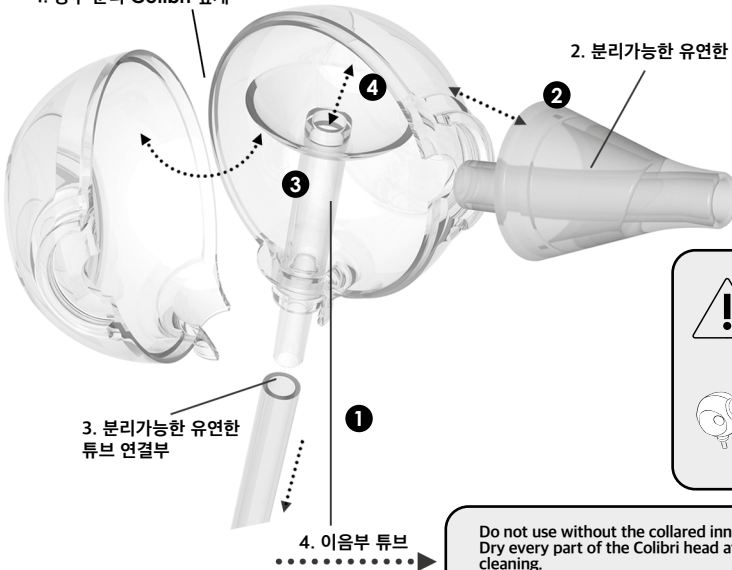
Use original Nosiboo parts! Our nasal  
aspirators work effectively only with  
original Nosiboo parts/accessories.

Do not use without the collared inner tube!  
Dry every part of the Colibri head after  
cleaning.

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# USER MANUAL

## Table of contents

Package contents	16
Foreword	16
Symbols	17
Important safety instructions	17
1. Intended use	19
2. Product introduction	19
3. First-time operation	20
4. Instructions for use	20
5. Troubleshooting malfunctions	21
6. Maintenance and storage	24
7. Cleaning	24
8. Important information about reducing the electromagnetic interferences	25
9. Environment	25
10. Specifications	25
11. Accessories, spare parts	26

**NOSIBOO PRO IS A HYGIENIC  
MEDICAL DEVICE**

**You may only exercise  
your right of withdrawal if the  
device is undamaged and  
the protective foil on the  
Colibri head is intact.**

**Guiding and applicable legislation:** Directive 2011/83/EU § 16 (e) of the European Parliament and of the Commission, Austrian Republic: FAGG: § 18 (1) 5, Hungary: 45/2014. (II. 26.) Government decree § 29 (1) (e).

## Package contents

\* Colibri head + newborn's nose tip \* Standard nose tip \* KOR Power cord \* User manual \* Device \* Tube \* Cleaning brush

## Foreword

Dear customer,

Congratulations on buying our KSCI 2020, KSCI 2021, KSCI 2022, PTPA, Red Dot Design and German Design awarded Nosiboo Pro nasal aspirator! Please read this user manual carefully so that you'll be able to use the nasal aspirator for years and can prevent illnesses for a long time.

**Please read the user manual carefully  
before using the device!**

**ATTRACT KFT. IS AN ISO 13485 CERTIFIED MEDICAL DEVICE  
MANUFACTURER. THIS PRODUCT IS AN MDD LEGACY DEVICE.**

**ACTOR EUDAMED NO.: HU-MF-000004252**

## Symbols

You may see the following signs in the user manual or on the label of the device:

### Explanation

Type BF applied parts  
(increased protection against  
electric shock)

Reference (order number)

Caution!

Class II device (double insulation)

Serial number

Hazardous waste

Class IIa medical device

Global Trade Item Number (UDI-DI)

Name and address of the  
manufacturer

Lot number (production identifier)

Read the user manual!

Protection against solid particles of di-  
ameter greater than 12.5 mm; protected  
from condensation.

For home use only

Keep dry!

EN

## Important safety instructions

**If the possible hazardous situations below are not avoided, user injury or even a fatal accident may occur.**

Safety instructions regarding general use

### A. General Precautions

- 1) The device can be used exclusively for removing the nasal mucus from a child's nose. It cannot be used for any other purpose!
- 2) It is FORBIDDEN to use the device in the bathroom or any other humid or wet environment!
- 3) Stop using the device if liquid or solid has been drawn into the housing. Contact a qualified service centre.
- 4) The device should only be used by adults.
- 5) While in use, the device should be in a stable position, standing on a flat surface.
- 6) Do not lay the child on their back and do not block the free nasal opening while using the device! Do not leave the device unattended near children or people with reduced physical or mental abilities!
- 7) Do not remove the mounting screws from the housing or reconfigure the device for any other purpose.
- 8) In the event of an unexpected failure or malfunction, immediately turn off the device and unplug the power cord from the power supply.
- 9) Keep the separated parts of the Colibri head out of reach of children.
- 10) The use of the device is generally intended for children under 6 years of age.

- 11) Keep the child in a stable position while using the device.
- 12) Do not block the ventilation holes on the back of the device during use.
- 13) Be careful as hot air may be discharged through the ventilation hole of the device.
- 14) Dissolve dry or thick nasal mucus using saline solution. The amount of saline solution used should not exceed 5 ml at one time.
- 15) After using saline solution, wait for a few minutes, and then use the device after removing the excess discharge from the nose.
- 16) After the power is turned off, the motor runs for a while to cool down the device. Be sure to unplug the power cord after the device has completely stopped working.
- 17) The use of the device is prohibited for patients with congenital malformations such as cleft palate, hemophilia, or in case of taking blood-thinning medicines.
- 18) The Accessory Set, a separately sold pack contains replaceable accessories used in this device. Its components must be used together with this device and shall not be used for any other purpose.
- 19) Use original Nosiboo parts! Our nasal aspirators work effectively only with original Nosiboo parts/accessories.
- 20) Do not aspirate water with the device! Do not use the nasal aspirator for nasal washing! Aspirating water with the electric device will result in loss of warranty.
- 21) The device cannot be used in oxygen-rich environments and in clinical circumstances! The medical device is for home use only.

## **B. Application precautions**

- 1) Do not use the device with the patient lying down.
- 2) Do not use the Colibri head and the device without the collared tube.
- 3) If mucus or saline solution is drawn into the tube, stop using the device and switch it off. Disconnect the tube from the device, empty the Colibri head, clean and dry the Colibri head and tube, and check the air mesh to replace it if it is wet or clogged.
- 4) Use the device only when the tube and Colibri head are dry.
- 5) Only use the tube and Colibri head supplied by the manufacturer of the device.
- 6) Do not use the Colibri head by connecting it to other equipment.
- 7) If the housing or power cord is damaged or cracked due to misuse, the use of the device is forbidden.
- 8) The recommended aspiration time is 10 seconds.
- 9) After the operation of the motor has completely stopped, you can restart the device.
- 10) If a rash or redness appears on the patient's nose after using the device, stop using the device and follow your doctor's instructions.
- 11) For patient safety and to prevent overuse, the device is automatically turned off after 14 minutes of operation.
- 12) In case of excessive use or insufficient ventilation, the power of the device is turned off by the operation of the thermal switch.

### *Safety instructions regarding connecting the device to the mains*

- \* Protect the power cord against damage! Do not break it, do not place heavy items on it because a damaged cord may cause electric shock.
- \* Keep the power cord away from heat (e.g. furnace, heater and convector) because it can cause damage to the isolation.
- \* Only use the device if the tube and the Colibri head are dry.
- \* Do not remove the mounting screws of the housing. Do not reconstruct the device for other purposes.
- \* The use of the device is forbidden if there is a breakage or crack on the housing or on the power cord due to misuse.
- \* In case of unexpected failure or malfunction, immediately switch off the device and interrupt the power supply by unplugging it from the mains.

### *Safety instructions regarding the next use of the device*

- \* Always disinfect the Colibri head after use!
- \* Do not leave the parts of the dismantled Colibri head within reach of children because they may swallow these.

### General precautions

- \* Read and follow the instructions of the "Important information about reducing the electromagnetic interferences" section!
- \* Read and follow the instructions of the "Environment" section if you would like to dispose of the device or any part of it.

## 1. Intended use

The product belongs to the category of medical electric devices intended to aspirate blood, body fluids, and acne. The Nosiboo Pro nasal aspirator was designed to remove the nasal mucus from the child's nose.

## 2. Product introduction

The vacuum and airflow needed for the aspiration is created by an aspirating device designed specifically for this purpose. The Colibri head, which is inserted into the nose, is connected to this device via a tube. The Colibri head has a patented design that prevents the mucus from entering the device.

### Intended purpose:

- \* to ensure nasal breathing to infants
- \* to prevent the complications of childhood common cold and to accelerate the healing process by aspirating the nasal mucus
- \* to relieve childhood allergy-like symptoms by removing the nasal mucus.

### Patients to be treated:

- \* children who are (typically under the age of 6) unable to remove the nasal mucus by blowing their noses.

### Illness, condition requiring treatment (indication):

- \* to eliminate clogging of the nose
- \* treatment of common cold and auxiliary treatment of superinfected paranasal sinuses
- \* supplementary treatment of the allergic infectious condition.

### Conditions in which the device cannot be used (contraindication):

- \* haemophilia
- \* if the blood vessels of the nasal septum's mucous membrane widen – in case of frequent nosebleed
- \* while taking blood-thinning medicines
- \* in case of congenital malformations (e.g. cleft palate).

### Suggested user:

- \* Age: adult over 18
- \* Sex: not applicable (man or woman)
- \* Knowledge: should have general knowledge according to their age
- \* Education: min. primary school education, no need for special knowledge to use the device.
- \* Important: knowledge of the User manual included in the package.

## 3. First-time operation

### *Assembling the Colibri head*

1. Clean and dry the parts of the Colibri head (see sections "Cleaning" and "Maintenance and storage").
2. Insert the collared tube into the nozzle housing that is supplied with a connection.
3. Place the nozzle member that has no connection in the right position.
4. Push the flexible nose tip to its place.
5. Attach one end of the tube to the Colibri head.

### *Attaching the device*

6. Make sure the air mesh behind the connecting unit is in place.
7. Attach the free end of the tube to the device housing.
8. Attach one end of the 220 V mains cord to the plug on the lower part of the device housing, then the other end to the socket.

The device is ready for use.

## 4. Instructions for use

### **A. Preparation before use**

- 1) Be sure to read the contents of the User Manual before using the device.
- 2) Check the housing, tube and Colibri head for damage. In case of damage, do not use the device and replace the damaged components.
- 3) Before using the device, check again whether the clean and dry collared tube is inserted into the Colibri head.
- 4) Before using the device, check that all components are properly connected.
- 5) Prior to use, the device should be positioned so that the power cord can be easily disconnected from the power outlet.
- 6) Before operating the device, check the appropriate air flow rate according to the age of the patient. The minimum setting is suitable for newborns, the medium setting is suitable for babies, and the maximum setting is suitable for young children (around 4 years old) and preschoolers (around 6 years old).
- 7) In case of the first use, prepare the device for use according to the procedure below.
  - A) Wash and dry the parts of the Colibri head.
  - B) Insert the collared tube into the nozzle housing and connect the other part of the nozzle housing.
  - C) Push the nose tip into its place.
  - D) Connect one end of the tube to the Colibri head.
  - E) Connect the other end of the tube to the device.
  - F) Connect the power cord to the device and the power outlet.

### **B. How to operate**

- 1) Press the power button to operate the device.
- 2) Turn the power button to properly adjust the suction strength. (Refer to Figure 2 "Adjusting the suction power")
- 3) Insert the nose tip of the Colibri head into the patient's nose and aspirate.
- 4) During use, the connection point of the Colibri head should face downward.
- 5) If you would like to finish aspiration, press the power button of the device to stop its operation.

### C. How to store and manage after use

- 1) Always disinfect the Colibri head after use.
- 2) Be careful not to damage the power cord by impact or heat.
- 3) Do not store equipment near heat sources.
- 4) Follow the method below for cleaning after use.
  - a) Remove the Colibri head from the tube, then the nose tip, and then the housing of the Colibri head.
  - b) Wash the tube, nose tip and housing of Colibri head with lukewarm water and detergent, and let them dry. It is not allowed to use a dishwasher, boiling water, microwave oven, vaporizer, or baby bottle sterilizer for cleaning the spare parts, because they might deform by heat!
  - c) Clean the housing of the equipment with a soft, dry cloth. Do not use liquids, detergents or chemicals.
  - d) If necessary, wipe the tube with a cloth and do not wash it.

## 5. Troubleshooting malfunctions

If you suspect malfunctioning of the Nosiboo Pro device, please read the next two chapters. As the first step, the Troubleshooting guide provides suggestions for resolving concrete problems. In the chart, you can find the problems and the troubleshooting suggestions. If these suggestions do not solve the problem, the Malfunction management guide helps you/your merchant/distributor in localizing the malfunction as soon as possible. Please study these chapters in detail in case of a malfunction.

The manufacturer shall not be held responsible for the failure of products from unauthorised import sources!

Troubleshooting guide

<b>Malfunction</b>	<b>Reason</b>	<b>Recommendation</b>
The device cannot be switched on.	The plug is not connected.	Connect the power cord to the device then to the 220 V socket.
The device cannot be switched on.	The device switched off due to overuse for security reasons.	For the sake of your child's health, stop using the nasal aspirator. The device can be used again in appr. 50 minutes.
The suction intensity decreased.	The Colibri head got blocked or was incorrectly assembled.	Take it apart and clean it. After drying it, assemble it based on the user manual.
The suction intensity decreased.	The air mesh wore out.	Replace the air mesh based on the user manual's „Maintenance and storage" section.
Liquid or mucus got into the tube.	The Colibri head was not held in the correct position. The connection point of the tube did not point downwards.	Switch off the device and clean and dry the Colibri head and the tube. Check the air mesh and replace it if necessary.
Liquid or mucus got into the tube.	Too much saline solution was used.	Reduce the volume of the saline solution to less than 5 ml. Switch off the device and clean it, then dry the Colibri head and the tube. Check the air mesh and replace it if necessary.

If the device does not work properly in spite of performing the instructions of the troubleshooting guide, check the Malfunction management guide with the distributor/merchant.

*Malfunction management guide*

*The guarantee is valid for 24 months after the purchase date. It will be only validated if the device was used according to its intended purpose and does not bear signs of damage. In the next step the existence of malfunctioning has to be cleared. And if the malfunction occurred due to the user's own fault or if it was no fault of his own. The manufacturer provides 24 months warranty in case of use according to the intended purpose. The following checks can be conducted by any user at home or with the distributing merchant or their trustee after presenting the filled in warranty card or the receipt. The distributor or the legal entity charged with failure assessment is obliged to fill in the warranty card supplied with the User manual or provide their own warranty document, record the steps of troubleshooting in detail and forward it to the manufacturer within 24 hours.*

## Troubleshooting

## Detailed description

## Warranty tasks

### Visual inspection

The entireness of the label on the bottom of the device.

Signs of damage on the label indicate opening of the device housing.

Due to misuse of the device, the warranty is not valid.

Other signs of damage on the housing, scratches, bumps, bruises, strains, scratches around the screws.

Signs of damage on the device housing indicate opening of the device housing.

Due to misuse of the device, the warranty is not valid.

Traces of contamination on or around the air mesh. Damage of the opening behind the air mesh, grit in the device or traces indicating that grit has been drawn into the device.

The excessive contamination of the air mesh, dirt getting into the device indicate the misuse of the device.

Due to misuse of the device, the warranty is not valid.

Any kind of contamination of the tube

Contamination getting into the tube indicates misuse of the device.

Due to misuse of the device, the warranty is not valid.

### Manual inspection

Before operating the device, check the connection of the power cord on the device.

If the connection (not the power cord) on the device caved in, can be moved, can be removed, turning on and operating the device is life-threatening! Such malfunctions are due to external impact and are not covered by the warranty.

Due to misuse of the device, the warranty is not valid.

Any kind of failure on the control button.

If the control button on top of the device is damaged, is jammed, moves freely or fell off, the malfunction is due to external impact and is not covered by the warranty. Turning on and operating the device is life-threatening!

Due to misuse of the device, the warranty is not valid

### Powering up/turning on

Powering up the device happens with the power cord supplied with the device.

If the device does not start to operate under these conditions, the replacement of the power cord is suggested. Such malfunctions are due to external impacts and are not covered by the warranty.

Due to misuse of the device, the warranty is not valid.

Powering up the device is possible but after turning on the device irregularity occurs.

If you experience "unusual" phenomena while operating the device, e.g. low suction power, smoke emission, strong smell, the device buzzes or clacks unusually loud, it shakes, it shorted out, cannot be adjusted via the control button and does not turn off after 15 minutes.

**If you experience "unusual" phenomena while operating the device and the device does not bear traces of physical damage included in the first two points, the customer's device is eligible for warranty.**

### Warranty tasks

The distributor of the Nosiboo Pro device is obliged to promptly handle customer complaints based on the aspects included in the chapters Troubleshooting guide and Malfunction management guide of the User manual. The distributor of the device is obliged to keep a written record about the troubleshooting. This can be done within the framework of the distributor's internal documents or in the warranty card supplied with the User manual. It has to contain the detected malfunction, the date and the unique serial number to be found on the label on the bottom of the device.

### Specialist service station contact details

JD International  
Phone: 02-419-3927  
(Mon-Fri: 10am-4pm / Lunch 12pm-1pm)  
e-mail: customer@nosiboo.kr  
web: www.nosiboo.kr

## 6. Maintenance and storage

Only after the cooling process has stopped should the power cord be unplugged from the mains and from the device! Disconnect the tube from the device and the Colibri head. There is an air mesh behind the connecting unit which prevents solid objects and dust from entering the device. In the event of an unexpected drop in performance, it is recommended that you check the air mesh. If the air mesh is dirty or clogged, replace it with a spare air mesh. (A list of spare parts can be found in the "Accessories, spare parts" section.)

### **Figure 1 – Replacing the air mesh**

Clean the housing with a dry, soft cloth; do not use any detergents or chemicals!

Keep the device in a closed space where it is protected from dust and damage (e.g. in a cabinet).

If you have more children, we recommended to use a separate Colibri head for each child and to store the Colibri heads separated.

Do not store near a heat source!

## 7. Cleaning

1. Remove the tube and the nose tip, then take apart the head housing.
2. Wash the collared tube, the nose tip, and the head housing in lukewarm water and dishwashing detergent and let them dry.

### **Figure 4 – Cleaning of the Colibri head**

3. Clean the device housing with a dry, soft cloth; do not use any liquids, detergents or chemicals.
4. Wipe the tube with a cloth if necessary. Don't wash the tube.

## 8. Important information about reducing electromagnetic interferences

The home wireless network equipment, mobile phones and wireless phones may affect the operation of medical equipment, as electromagnetic interference can occur between devices. Electromagnetic interference may cause harmful interference in the operation of medical devices and therefore emergency situations may occur.

Due to the above mentioned reasons, medical devices mustn't interfere with any other equipment.

To guarantee the electromagnetic compatibility, the necessary requirements were fixed in standards. The standards define the protection level of the medical device and the maximum value of the electromagnetic radiation emitted by these.

The Nosiboo Pro nasal aspirator complies with the requirements of EN 60601-1-2:2017. In spite of that, you should follow the instructions below while using the device:

- \* Do not use your cell phone or wireless phone or any other device emitting strong electromagnetic radiation near the device! Otherwise, the device may malfunction which can cause an emergency.
- \* It is recommended to keep at least 5 m distance between the devices! Check the proper operation of the device if this distance is shorter!

## 9. Environment

Do not dispose of the device at the end of the product life with household waste because electronic products qualify as hazardous waste. This is indicated by the following sign on the product packaging:

The Colibri head does not present a serious risk to the environment so it can be disposed of as household waste.

## 10. Specifications

Max. motor performance: 500 W  
 Adjustable motor performance: 150-390 W  
 Performance: 420 VA  
 Differential pressure: 0.04-0.08 bar  
 Max. air flow: 30-42 l/min  
 Power supply: 220 V 60 Hz  
 IP class: IP21  
 Protection class: II  
 Type of applied parts: BF  
 Applied parts: the complete housing  
 Operation type: no continuous operation  
 Operation length: on 15 min/appr. off 50 min  
 Duration of continuous use: max 15 min  
 Safety switch release time: after 15 minutes  
 Tripping temperature of the thermal switch: 95 °C  
 Average housing temperature: 30-35 °C after 1 min continuous use  
 Noise level: 60-70 dB  
 Expected life of the housing (electrical parts): 3 years  
 Expected life of the Colibri head: 3 months  
 Diameter of the nasal insertion end of the standard nose tip: max. 7.0 mm

Length of the nasal insertion end of the standard nose tip: max. 8.0 mm  
 Diameter of the nasal insertion end of the newborn's nose tip: max. 4.9 mm  
 Length of the nasal insertion end of the newborn's nose tip: max. 12.6 mm  
 Weight: 1090 g  
 Dimensions (housing, W x H x D): 198 x 162 x 172 mm  
 Dimensions (complete package): 180 x 180 x 225 mm  
 Operating conditions:  
 1. Temperature range: -5 °C to +40 °C  
 2. Relative humidity: max. 75%  
 3. Atmospheric pressure / barometric pressure range: 525-795 mmHg / 70-106 kPa  
 Transportation conditions:  
 1. Temperature range: -20 °C to +65 °C  
 2. Relative humidity: 10-75%  
 3. Atmospheric pressure / barometric pressure range: 150-795 mmHg / 20-106 kPa  
 Storage conditions:  
 1. Temperature range: -20 °C to +65 °C  
 2. Relative humidity: 10-75%  
 3. Atmospheric pressure / barometric pressure range: 525-795 mmHg / 70-106 kPa

## 11. Accessories, spare parts

### Nosiboo Pro Accessory Set:

Accessory pack consisting of Nosiboo Pro spare parts

ACS-01-01 blue

- \* 1 Colibri head
- + standard nose tip
- \* 1 newborn's nose tip
- \* 1 tube
- \* 1 air mesh
- \* 1 cleaning brush

ACS-01-04 grey

- \* 1 Colibri head
- + standard nose tip
- \* 1 newborn's nose tip
- \* 1 tube
- \* 1 air mesh
- \* 1 cleaning brush

ACS-01-02 pink

- \* 1 Colibri head
- + standard nose tip
- \* 1 newborn's nose tip
- \* 1 tube
- \* 1 air mesh
- \* 1 cleaning brush

ACS-01-13 sage

- \* 1 Colibri head
- + standard nose tip
- \* 1 newborn's nose tip
- \* 1 tube
- \* 1 air mesh
- \* 1 cleaning brush

ACS-01-03 green

- \* 1 Colibri head
- + standard nose tip
- \* 1 newborn's nose tip
- \* 1 tube
- \* 1 air mesh
- \* 1 cleaning brush

### Official distributor in the Republic of Korea

Nosiboo Korea (JD International) Phone: 02-419-3927 (Mon-Fri; Room 95, 10th floor, Testa Tower, 25 Misagangbyeonseo-ro, Hanam-si, Gyeonggi-do; ZIP code: 12918)  
Home page: [www.nosiboo.kr](http://www.nosiboo.kr)  
customer@nosiboo.kr

Power cord KR  
300052

Connecting unit  
300033 \* blue  
300034 \* pink  
300035 \* green  
300058 \* sage  
300059 \* grey

The guarantee is valid for 24 months after the purchase date. It will be only validated if the device was used according to its intended purpose and does not bear signs of damage. Warranty is validated at the authorised distributor of the device. (The authentic distributor invoice has to be presented.) The distributor of the Nosiboo Pro device is obliged to promptly handle customer complaints based on the aspects included in chapters Troubleshooting guide and Malfunction management guide of the User manual. The distributor of the device is obliged to keep a written record about the troubleshooting. This can be done within the framework of the distributor's internal documentation or in the warranty card supplied with the User manual. It has to contain the perceived malfunction, the date and the unique serial number to be found on the label on the bottom of the device.

**To be completed by the distributor**

Name and address of distributor:

Serial number of the Nosiboo Pro nasal aspirator:  
(to be found on the bottom of the device, on the label)

Purchase date:

Type of malfunction:

Date of malfunction:

Countersignature of distributor:  
(date, signature and seal)

**To be completed by the service**

Date of receiving:

Serviced parts:

New serial number:

Date of returning:

Countersignature and seal of service:

Warranty period of the replaced parts is 24 months.

**To be completed by the distributor**

Name and address of distributor:

Serial number of the Nosiboo Pro nasal aspirator:  
(to be found on the bottom of the device, on the label)

Purchase date:

Type of malfunction:

Date of malfunction:

Countersignature of distributor:  
(date, signature and seal)

**To be completed by the service**

Date of receiving:

Serviced parts:

New serial number:

Date of returning:

Countersignature and seal of service:

Warranty period of the replaced parts is 24 months.

*The expressed warranty does not affect the consumer's implied warranty rights!*

**Nosiboo Pro is a hygienic medical device. You may only exercise your right of withdrawal if the device is undamaged and the protective foil on the nose tip is intact. Due to hygienic reasons, we do not replace the device if the packaging has been opened.** Guiding and applicable legislation: Directive 2011/83/EU § 16 (e) of the European Parliament and of the Commission, Austrian Republic: FAGG: § 18 (1) 5, Hungary: 45/2014. (II. 26.) Government decree § 29 (1) (e)

**Manufacturer:** ATTRACT Kft. Hungary, H-7622 Pécs, Siklósi út 1/1.

ISO 13485:2016 Certified

**Official distributor in the Republic of Korea**

**Tel: 02-419-3927 (Monday-Friday: 10 a.m. - 4 p.m. / Lunch: 12 p.m. - 1 p.m.)**

**E-mail: customer@nosiboo.kr**

nosiboo<sup>\*</sup>pro<sup>®</sup>  
electric nasal aspirator  
[www.nosiboo.kr](http://www.nosiboo.kr)